

A GAME CHANGING APPLICATION FOR DRY SOCKETS*

ORCA® COMPLETE DRY SOCKET

for dry sockets and oral pain

Non-Medicated • Only Natural Ingredients • Clinically Proven



**Designed and Packaged for Oral Surgery
and Dental Applications**

Highly Effective, Non-Medicated Alternative, Consists
only of Essential Oils and other Proprietary Natural
Ingredients

Outperforms Other Products Hands Down!

Clinically used in Oral and Maxillofacial Surgery practice
for over 16 years with thousands of applications. Over
95% of patients experience complete resolution of dry
socket symptoms with one single treatment

Simple and Effective Delivery


Recommended to be applied to socket or extraction site
with ORCA Foam™ which will dissolve within a week
without the need for removal. Delivery methods may
also include an alternative carrier that will dissolve or
gauze strip (removal required)



Directions: For adults and children 12 years of age and older. In a sterile fashion, place
5 to 8 drops onto ORCA® Foam, similar resorbable gelatin sponge or gauze strip. Apply
directly to dry socket or areas of discomfort in oral region.

FOR PROFESSIONAL USE ONLY


STEP 1:
IRRIGATE THE WOUND
WITH SALINE

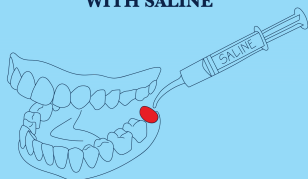

STEP 2:
SATURATE ORCA FOAM
USE: 5 TO 10 DROPS OF
ORCA COMPLETE DRY SOCKET



STEP 3:
PLACE ORCA FOAM
IN WOUND

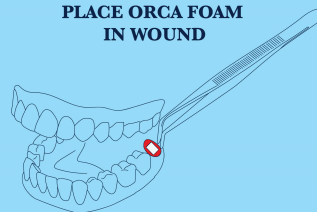

ORCA®
PROFESSIONAL

Innovation. Integrity. Trust.

**ORCA FOAM WILL DISSOLVE IN
APPROXIMATELY 7 DAYS.
PATIENT SHOULD NOT IRRIGATE
THE WOUND FOR 3 DAYS.**

*The founder of ORCA has had years of referrals from colleagues that have had dry sockets which were refractory to other products and ORCA Professional has always risen to the occasion to resolve discomfort!
For more information: www.orca-products.com